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visual card

For in vitro medical device
31027 - 10 test

Rapid test on card for cocaine detection in urine

PRINCIPLE

Cocaine extracted from leaves of coca plant is a potent central nervous system stimulant and a local anesthetic.

Cocaine induce euphoria and exaltation sense, accompanied by increased heart rate, dilation of the pupils, fever, tremors and sweating.

Cocaine is taken by smoke, intravenous, intranasal and oral and is eliminated by urine as benzoylecgonine in short time. Benzoylecgonine has a biological half-life of 5-8 hours, which is much longer than that of cocaine (0.5-1.5 hour), and can be generally detected for 24-60 hours after cocaine use or exposure.

The test use an immunometric "competitive" method for benzoylecgonine detection by competition between benzoylecgonine fixed to the membrane and benzoylecgonine free, eventually present in the tested sample, for the conjugate anti-cocaine-colouring. A sufficient quantity of cocaine metabolites, above cut-off levels, will occupy all binding sites of antibodies and will prevent the bond between the conjugate anti-cocaine-colouring and the coloured band formation on the membrane. Therefore, the formation of one coloured band shows a positive assay. Instead, the contemporaneous appearance of two coloured bands indicates a negative result. This method represents a valid, simple and fast alternative to classic methods as gas chromatography (GC) or mass spectrometry (MS) which require complicated equipment and longer execution times. However, this assay provides only a preliminary analytical test result and, in case of positivity, it must be confirmed by other chemical researches.

REAGENTS

Cards (COC)	10
Specimens pipettes	10

STORAGE AND STABILITY

Cards are stable up to expiry date showed on the package, stored at room temperature (15-25°C) or refrigerated (4-8°C) .

PREPARATION OF REAGENTS

Reagents are ready to use.

SAMPLE

Fresh urine.

QUALITY CONTROLS

Good laboratory practice recommends the use of control materials to ensure proper kit performances. When testing the positive and negative controls, use the same assay procedure as with a urine specimen.

PROCEDURE


- Allow specimens and cards to come to room temperature prior to testing.
- Fill the urine dropper with specimens and, by holding it vertically, dispense 3 drops (150 µl) of urine in the sample well A of the card.
- Wait 3-8 minutes to read the results. Do not read after 8 minutes because the intensity of the coloured bands may change, or a new line may appear which was no present at the end of the incubation time (5 minutes). Therefore, it's most important to record the results of assay immediately after the five minutes incubation period.

Prudences


- Urine specimens and all materials coming in contact with them should be handled and disposed as if it is infectious and capable of transmitting infection. Avoid contact with skin.
- Avoid cross-contamination of urine samples by using a new specimen cup and specimen pipette for each urine sample.
- Test the sample as soon as possible, preferably the same day of specimen collection.
- The urines may be refrigerated at 2-8°C for 2 days or frozen at -20°C for longer periods.
- Specimen that were refrigerated must reach room temperature (15-25°C) prior to testing.
- Specimen previously frozen must be thawed and mixed thoroughly prior to testing.

INTERPRETATION OF RESULTS

POSITIVE: One colour band appears on the control window (C). The cocaine and/or metabolites concentration is over cut-off levels.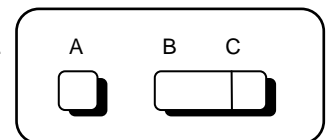

NEGATIVE: Two colour bands appear, even if with weak intensity. The cocaine and/or metabolites concentration is under cut-off levels.

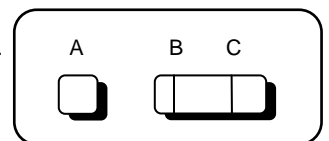ERROR: If there is no distinct colour band visible both in the test window and test control window the test is inconclusive. It is recommended to repeat the test.

Note

- A very faint line on the test region indicates that the cocaine in the sample is near or below the cut-off level for the test. However, if any line is observed at the test region, the sample is negative.
- Confirm eventually positive results by opportune confirmation assay.

ANALYTICAL PERFORMANCES

Specificity

- The test is specific for benzoylecgonine (cocaine metabolite in urine).
- The test specificity was tested by adding various drugs, drug metabolites, and other compounds that are likely to be present in urine. All compounds were prepared in drug-free normal human urine.

Interferences

- The following compounds produce positive results when tested at levels equal to or greater than the concentrations listed below.

Compound	Concentration (ng/ml)
Benzoylecgonine	300

- The following compounds were found not to cross-react when tested at concentrations up to 100 µg/ml:

4-Acetaminophenol	(+ -)Chlorpheniramine
Acetylsalicylic Acid	Naloxone
Ampicillin	Dextromethorphan
Amitriptyline	4-Dimethylaminoantipyrine
L-Ascorbic Acid (vitamina C)	(+)Epinephryne
Asp-phe Methyl Ester	Ranitidine
Atropine	Caffeine
Ethyl P-Aminobenzoate	Procaine
Chloroquine	Penicillin G
Imipramine	(+)Phenylpropanolamine
(+ -)Isoproterenol	B-Phenylethylamine
Lidocaine	Sulindac
Quinidine	Erythromycin
3-Hydroxytyramine	Naltrexone
Furosemide	Tyramine
(S)-6-Methoxy-2Methyl-2-Naphthaleneacetic Acid	

Correlation-accuracy

The comparison between the results obtained by the kit cocaine easy-card and a commercial immunoenzymatic kit on 72 positive samples and 163 negative samples, gives a recovery of 100%. The correlation on 77 samples, resulted positives by GC/MS assays, gives a recovery of > 99%.

Sensitivity

The test sensitivity is ≥ 300 ng/ml.

LIMITATIONS OF PROCEDURE

- The assay is designed for use with human urine only.
- A positive result with any of the tests indicates only the presence of a drug/metabolite and does not indicate or measure intoxication.
- There is a possibility that other substances and/or factors not listed may interfere with the test and cause false results, such as technical or procedural errors. See "Specificity" for lists of substances that will produce positive results or that do not interfere with tests performance.

WASTE MANAGEMENT

Please refer to the local legal requirements.

Note

The number showed on single product refer to the production lot. The number showed on the exterior label refer to the packaging. In case of claims, please refer to one of these two number.

BIBLIOGRAPHY

Baselt, R.C. Disposition of toxic Drugs and Chemical in Man, Davis, Biomedical Communication, 1982.
Urine teting for Drugs of Abuse, National Institute on Drugs Abuse (NIDA), Research Monograph 73, 1986.
Fed. Register, Department of Health and Human Services, mandatory Guidelines for Federal Workplace Drug Testing Programs, 53, 69, 11970-11979, 1988.
McBay, A.J. Clin. Chem. 33, 33B-40B, 1987.
The Pharmacological Basis of Therapeutics, Gilman, A.G. Goodman, L.S. and Gilman A. eds. MacMillan Publishing, New York, NY, 1980.

SYMBOLS

Read instruction for use

CE mark (requirement of 98/79 regulation)

Storing temperature limits

In vitro medical device

Producer